

# SHANTO DENTAL



## LABORATORY

---

As a Full Service Digital Dental Laboratory, Shanto Dental Laboratory accepts files from all scanners. Find your scanner below, and follow the instructions, in order to start sending us your cases today!

## MEDIT Link

---

- Go to [www.meditLink.com](http://www.meditLink.com)
- Log in with your username and password.
- Click on Partners  search [info@shantodental.com](mailto:info@shantodental.com).
- Send a request by clicking 

Once we accept your request you will be able to “order” your case after scanning.

## iTero

---

- Call iTero support at 1.800.577.8767
- Select option 1, then option 1
- Request to have Shanto Dental Laboratory, added to your scanner
- Provide our lab name and the iTero customer number 4757

On the scanner—go to settings and select “Sync Configuration”

## Carestream

---

- Visit Carestream connect on your scanner
- Search for Shanto Dental Laboratory (info@shantodental.com)
- Add Lab
- Select Shanto Dental Laboratory when submitting scans or raw .STL files
- Email your favorite digital lab, Shanto Dental Laboratory

- Useful link:

[3Shape TRIOS - Making connections - YouTube](#)

- Log in to 3Shape software or link below:

<https://us.3shapecommunicate.com>

- Click "More" and select settings

- Then in the menu click "Connections"

- Click on "Labs" then the "Add"

- Search Shanto Dental Laboratory by email address: [info@shantodental.com](mailto:info@shantodental.com)

- Click "Connect" to send a connection request

- We will approve your request ASAP

- Once we approved your connection request it will be added to your list of connected Labs

- Click on "Patients" and "Save" your changes

- Log in to Cerec Connect at [www.sirona-connect.com](http://www.sirona-connect.com)

- Select "My Cerec Connect" then "Edit Account"

- Under "My Account" select "My Favorite Laboratories"

- Type in name Shanto Dental & DigitalOneDental ([info@shantodental.com](mailto:info@shantodental.com)) in the field for "Company name" then the Postal Code

- Click "Find"

- Check the box beside Shanto Dental & DigitalOneDental

- Click "ADD" to complete the registration

---

Vancouver  
T | 604.738.7317  
1445 Charles Street, Vancouver, BC V5L 2S7

Okanagan  
T | 778.738.6546  
212 - 2544 Enterprise Way, Kelowna, BC V1X 7X5

[shantodental.com](http://shantodental.com)  
[info@shantodental.com](mailto:info@shantodental.com)

